

## INSTRUCTIONS FOR USE

# 3-piece soft fibre mattress



## Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

## CE Marking

The 3-piece soft fiber mattress is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With the CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report any accident or incident involving our products to COMCO AB.

## Unpacking

Our mattresses are delivered roll packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position for about 1-3 hours to recover its normal original shape. The mattress can also be used immediately after unpacking.

## Introduction & Use

COMCO's 3-piece soft fibre mattress is held together by a cover and placed on top of an existing mattress. It is suitable for use in both healthcare and home environments. Each part of the mattress is filled with siliconised polyester hollow fibre. The properties of the smooth fibres provide a comfortable feel, reduce the risk of shearing and help to relieve aches and pains.

## Facts 3-piece soft fibre mattress

**Filling:** Siliconised polyester hollow fibre

**Packaging:** Roll packed in clear plastic

**Recommended user weight:** 0-95 kg

**Colour:** White

Label with instructions for washing, user weight, article number, size, and date of manufacture is placed on all parts of the mattress.

## Sizes

90x200 cm, article no. 3COM090200L

105x200 cm, article no. 3COM105200L

120x200 cm, article no. 3COM120200L

Special sizes can also be produced on request.

## Accessories

**The hygiene cover** is available as an accessory for the 3-piece soft fibre mattress. The cover is vapour permeable, but does not let moisture/body fluids through. It is washable at 90°C and can also be tumble dried.

**The mattress protector/hygiene cover is available in the following sizes:**

80x200cm, article no. HM-080200

90x200cm, article no. HM-090200

105x200cm, article no. HM-105200

120x200cm, article no. HM-120200

Special sizes can also be produced on request.

## Washing the mattress

The mattress is washed at 60°C and can also be tumble-dried on low heat to iron dry.

## Destruction

Both mattress and packaging are sorted under combustible waste.

## Returns and complaints

All deviations must be reported to COMCO AB. Complaints may only be made when the deviation is reported to us. Return is then done according to our instructions. If the goods have been damaged in transit, the transport company must also be notified. When the complaint is received by COMCO, the customer will be informed of how it will be dealt with.

## Warranty

**Warranty:** 2 years

## Other information

The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period of time, we recommend storing it flat.

## Manufacturer

### Contact

**Phone:** +46 (0)40 45 47 87

**Email:** info@comco.se

**Website:** www.comco.se

### Support

Weekdays without holidays 8.00-17.00

### Postal address

COMCO AB  
Brädgårdsvägen 28  
236 32 Höllviken  
SWEDEN

### Freight address

COMCO AB  
Radiatorvägen, Port 5-8  
283 43 Osby  
SWEDEN