

INSTRUCTIONS FOR USE

Soft fibre mattress



Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

CE Marking

The Soft fibre mattress is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With the CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report any accident or incident involving our products to COMCO AB.

Unpacking

Our mattresses are delivered roll packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position for about 1-3 hours to recover its normal original shape. The mattress can also be used immediately after unpacking.

Introduction & Use

Filled with siliconised polyester hollow fibres, the soft fibre mattress offers good comfort and pressure relief. The smooth fibres provide a pleasant feel and reduce the risk of shearing. Suitable for both healthcare and home environments, it is placed on top of another mattress and secured by elasticated straps at each corner. The bed flaps fold under for better stability.

Facts Soft fibre mattress

Filling: Siliconised polyester hollow fibre

Fabric: Microfibre 104 g/m²

Packaging: Roll packed in clear plastic

Recommended user weight: 0-75 kg

Colour: White

A label with the supplier's name, instructions for washing, user weight, article number, size and date of manufacture is placed on the mattress.

The mattress is Oeko-tex certified.

Sizes

80x200 cm, article no STCOM080200

90x200 cm, article no STCOM090200

105x200 cm, article no STCOM105200

120x200 cm, article no STCOM120200

Special sizes can also be produced on request.

Accessories

The hygiene cover is available as an accessory for the soft fibre mattress. The cover is vapour permeable, but does not let moisture/body fluids through. It is washable at 90°C and can also be tumble dried.

The mattress protector/hygiene cover is available in the following sizes:

80x200 cm, article no. HM-080200

90x200 cm, article no. HM-090200
105x200 cm, article no. HM-105200
120x200 cm, article no. HM-120200
Special sizes can also be produced on request.

Cleaning

The mattress is washed at 60°C (can be washed on a full-length programme in a domestic washing machine) and can be tumble dried on a low heat to dry. The cover can be washed at up to 90°C and can also be tumble dried.

Destruction

Both mattress and packaging are sorted under combustible waste.

Returns and complaints

All deviations must be reported to COMCO AB. Complaints may only be made when the deviation is reported to us. Return is then done according to our instructions. If the goods have been damaged in transit, the transport company must also be notified. When the complaint is received by COMCO, the customer will be informed of how it will be dealt with.

Warranty

The soft fibre mattress has a 2-year warranty.

Other information

The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period of time, we recommend storing it flat.

Manufacturer

Contact

Phone: +46 (0)40 45 47 87

Email: info@comco.se

Website: www.comco.se

Support

Weekdays without holidays 8.00-17.00

Postal address

COMCO AB
Brädgårdsvägen 28
236 32 Höllviken
SWEDEN

Freight address

COMCO AB
Radiatorvägen, Port 5-8
283 43 Osby
SWEDEN