


INSTRUCTIONS FOR USE

Solatium Base mattress

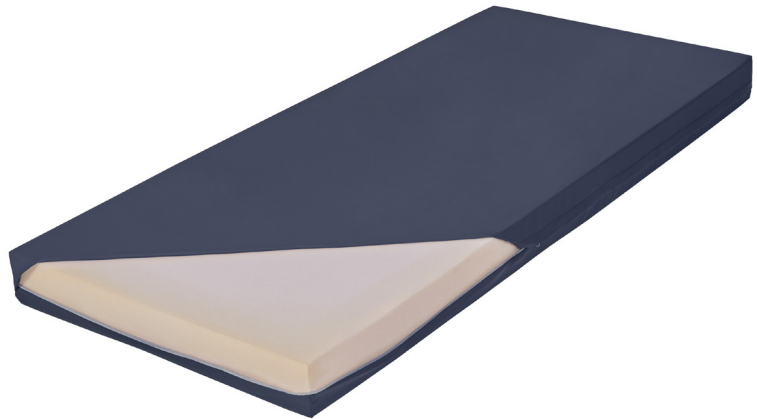

Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

CE marking

Solatium Base mattress is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report to COMCO AB if an accident or incident involving our products occurs.

Unpacking

Our mattresses are delivered roll-packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position to recover its normal original shape. The mattress can also be used immediately after unpacking.

Introduction & Use

The Solatium Base mattress is placed directly on the bed base and is suitable for both care and home environments. It provides good comfort and relief and is mainly used by users with varying degrees of disability. The removable hygiene cover has a zip with anti-drip protection on three sides and is durable, wipeable and equipped with carrying handles.

Facts Solatium Base mattress

Filling: 12 cm Profiled HR foam 35 kg/m³ (HR3532)

Hygiene cover: 200 g/m²

Packaging: Roll-packed in clear plastic

Recommended user weight: 0-130 kg

Colour: Navy blue

A label with instructions for washing, user weight, article number, article name, size and date of manufacture is placed on the inside of the hygiene case. The mattress is Oeko-tex certified and has an estimated lifetime of five years.

Sizes

80x200 cm, article no. S3512-080200

85x200 cm, article no. S3512-085200

90x200 cm, article no. S3512-090200

90x210 cm, article no. S3512-090210

105x200 cm, article no. S3512-105200

120x200 cm, article no. S3512-120200

Special sizes can also be made to order.

Accessories

The evacuation cover is available as an accessory for the Solatium Base mattress. This functional cover has Velcro straps that can be easily pulled out from each long side to secure the user in an emergency, and handles to facilitate evacuation/transfer.

The product is design protected through the European Union Intellectual Property Office (EUIPO).

The evacuation cover is available in the following sizes:

85x200cm, article no. TMEV-S12-085200
90x200cm, article no. TMEV-S12-090200
105x200cm, article no. TMEV-S12-105200

Special sizes can also be made to order.

Cleaning and reconditioning

The mattress hygiene cover should be cleaned and reconditioned to avoid the possible spread of infection. It is washed at a maximum of 95°C and can also be tumble dried. A label with washing instructions is placed on the inside of the case.

Use a damp cloth with a gentle detergent to clean the surface. Then wipe the damp surface with a dry cloth. If necessary, also use disinfectants such as DAX, Virkon, Perform or other commonly available disinfectants. The cover can be cleaned with alcohol-based surface disinfectants containing surfactants. When cleaning with chlorine, a concentration of 1 000 ppm/0.1% can be used.

After cleaning, check the cover. If it is damaged, it should be replaced. In this case, also check that no liquid has penetrated to the mattress core.

Destruction

Both mattress and packaging are sorted under combustible waste.

Returns and complaints

All discrepancies must be reported to COMCO AB. Complaints may only be made when the deviation has been reported to us. Returns are then made according to our instructions. In the case of goods damaged in transit, notification must also be made to the transport company concerned. When the complaint is received by COMCO, feedback is given to the customer regarding how the complaint should be resolved.

Warranty

Warranty: 3 years

Other information

The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period of time, we recommend storing it flat.

Manufacturer

Contact

Phone: 040 45 47 87

Email: info@comco.se

Website: www.comco.se

Support

Weekdays without holidays 8.00-17.00

Postal address

COMCO AB
Brädgårdsvägen 28
236 32 Höllviken, Sweden.

Freight address

COMCO AB
Radiatorvägen, Port 5-8
283 43 Osby, Sweden.

