

## INSTRUCTIONS FOR USE

# Solatium Heavy



## Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

## CE Marking

Solatium Heavy is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With the CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report any accident or incident involving our products to COMCO AB.

## Unpacking

Our mattresses are delivered roll packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position to recover its normal original shape. The mattress can also be used immediately after unpacking.

## Introduction & Use

Solatium Heavy is used in care environments and is adapted for heavier users. The mattress is placed directly on the bed base. The removable, durable hygiene cover has a zipper with anti-drip protection on three sides and is wipeable and equipped with carrying handles.

## Facts Solatium Heavy

**Filling:** 15 cm foam 55 kg/m<sup>3</sup>

**Cover:** Hygiene cover 200 g/m<sup>2</sup>

**Packaging:** Roll packed in clear plastic

**Recommended user weight:** 0-250 kg

**Colour:** Navy blue

A label with instructions for washing, user weight, article number, article name, size and date of manufacture is placed on the inside of the hygiene case.

The mattress is Oeko-tex certified.

## Sizes

90x200x15 cm, article no. HESO-09020015

90x210x15 cm, article no. HESO-09021015

105x200x15 cm, article no. HESO-10520015

105x210x15 cm, article no. HESO-10521015

120x200x15 cm, article no. HESO-12020015

Special sizes can also be produced on request.

## Cleaning

The mattress cover should be cleaned and reconditioned to avoid any possible spread of infection. Label with washing instructions is placed on the inside of the case. Hygiene covers are washed at a maximum of 95°C and can also be tumble dried.

Use a damp cloth with a gentle detergent to clean the surface. Then wipe the damp surface with a dry cloth. If necessary, also use disinfectant.

## **Destruction**

Both mattress and packaging are sorted under combustible waste.

## **Returns and complaints**

All deviations must be reported to COMCO AB. Complaints may only be made when the deviation is reported to us. Return is then done according to our instructions. If the goods have been damaged in transit, the transport company must also be notified. When the complaint is received by COMCO, the customer will be informed of how it will be dealt with.

## **Warranty**

Solatium Heavy has a 2-year warranty.

## **Other information**

The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period of time, we recommend storing it flat.

## **Manufacturer**

### **Contact**

**Phone:** +46 (0)40 45 47 87

**Email:** [info@comco.se](mailto:info@comco.se)

**Website:** [www.comco.se](http://www.comco.se)

### **Support**

Weekdays without holidays 8.00-17.00

### **Postal address**

COMCO AB  
Brädgårdsvägen 28  
236 32 Höllviken  
SWEDEN

### **Freight address**

COMCO AB  
Radiatorvägen, Port 5-8  
283 43 Osby  
SWEDEN