


## INSTRUCTIONS FOR USE

# Solatium Spacer mattress

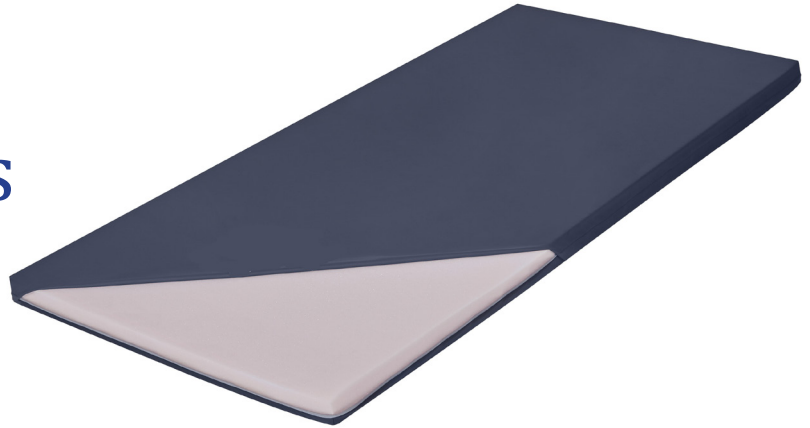

## Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

## CE marking

Solatium Spacer mattress is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report to COMCO AB if an accident or incident involving our products occurs.

## Unpacking

Our mattresses are delivered roll-packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position to

recover its normal original shape. The mattress can also be used immediately after unpacking.

## Introduction and use

The Solatium Spacer Mattress is used in combination with a preventive or therapeutic mattress and is placed underneath it. Its removable cover has a drip-proof zip on three sides and is both durable and wipeable. It is produced in a vapour-permeable material that does not let moisture/body fluids through and is elastic in two directions.

## Facts Solatium Spacer mattress

**Filling:** HR foam 35 kg/m<sup>3</sup>  
**Cover:** Hygiene cover 200 g/m<sup>2</sup>  
**Height:** 6 cm  
**Packaging:** Roll-packed in clear plastic  
**Recommended user weight:** 0-130 kg  
**Colour:** Navy blue

A label with instructions for washing, user weight, article number, article name and size is placed on the inside of the mattress cover.

## Sizes

80x200x6 cm, article no. SDM-3506-080200  
85x200x6 cm, article no. SDM-3506-085200  
90x200x6 cm, article no. SDM-3506-090200  
90x210x6 cm, article no. SDM-3506-090210  
105x200x6 cm, article no. SDM-3506-105200  
120x200x6 cm, article no. SDM-3506-120200

Special sizes can also be produced on request.

## Cleaning and reconditioning

The mattress cover should be cleaned and reconditioned to

avoid the possible spread of infection. A label with washing instructions is placed on the inside of the cover.

Hygiene covers are washed at a maximum of 95°C and can also be tumble dried. Use a damp cloth with a gentle detergent to clean the surface. Then wipe the damp surface with a dry cloth. If necessary, also use disinfectant.

## Destruction

Both mattress and packaging are sorted under combustible waste.

## Returns and complaints

All deviations must be reported to COMCO AB. Complaints may only be made when the deviation is reported to us. Return is then done according to our instructions. If the goods have been damaged in transit, the transport company must also be notified. When the complaint is received by COMCO, the customer will be informed of how it will be dealt with.

## Warranty

2 years warranty.

## Other information

The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period, we recommend that it is stored flat.

## Manufacturer

### Contact

**Phone:** +46 (0)40 45 47 87

**Email:** [info@comco.se](mailto:info@comco.se)

**Website:** [www.comco.se](http://www.comco.se)

### Support

Weekdays without holidays 8.00-17.00

### Postal address

COMCO AB  
Brädgårdsvägen 28  
236 32 Höllviken, Sweden.

### Freight address

COMCO AB  
Radiatorvägen, Port 5-8  
283 43 Osby, Sweden.

