


INSTRUCTIONS FOR USE

Stratum Sense Evacuation mattress

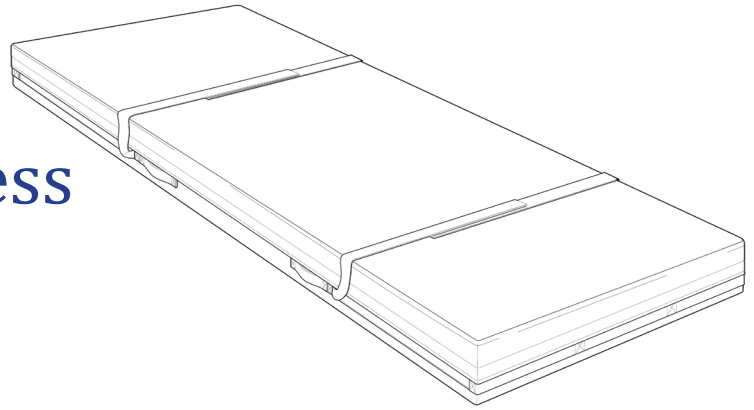

Safety instructions

- Read these instructions carefully and keep them for future reference.
- Do not insert sharp objects into the mattress.
- Only use the mattress for its intended purpose as instructed in this manual.
- Avoid injury by keeping the mattress away from hot sources or fire.
- Position the mattress carefully and check that the correct size is used.
- The mattress should be used in such a way that it is not damaged or crushed.

CE Marking

Stratum Sense Evacuation mattress is CE marked according to MDR (Medical Device Regulation) (EU) 2017/745 Medical Device Directive Class 1. With CE marking, COMCO AB ensures that the product meets the requirements of the Medical Device Act. According to these regulations, we are obliged to report accidents and incidents involving our products. We therefore ask you to report to COMCO AB if an accident or incident involving our products occurs.

Unpacking

Our mattresses are delivered roll-packed/compressed in recyclable plastic. Before starting to unpack, check that the mattress is not damaged. Then remove the plastic carefully. The use of sharp objects for opening should be avoided as there is a risk of damaging the mattress. After removing the plastic, the mattress should be placed in a flat position to recover its normal original shape. The mattress can also be used immediately after unpacking.

Introduction & Use

The Stratum Sense Evacuation mattress is a replacement mattress that is placed directly on the bed base. The mattress is equipped with Velcro straps that can be easily pulled out from each long side to secure the user in an emergency, and handles to facilitate evacuation/transfer. It is design protected through the European Union Intellectual Property Office (EUIPO).

The mattress is designed for the prevention and treatment of pressure ulcers up to category 2. It is divided into different zones with extra support when used in a raised position. The foam core is profiled, providing extra mobility. Foam and cover work together to prevent the negative impact of shear forces on the user.

Facts Stratum Sense Evacuation mattress

Foam core: Profiled high quality HR foam, 52 kg/m³

Cover: Evacuation cover 200 g/m²

Height: 14 cm

Packaging: Roll-packed in clear plastic

Recommended user weight: 0-150 kg

Colour: Navy blue

A label with the manufacturer's name, washing instructions, user weight, article number and name, size, and date of manufacture is placed on the inside of the cover. The top of the mattress/cover is printed with the manufacturer's name, product name, position marking, item number, size, year of manufacture, user weight and serial number.

Sizes

85x200 cm, article no. EHR52-085200-14

85x210 cm, article no. EHR52-085210-14

90x200 cm, article no. EHR52-090200-14

90x210 cm, article no. EHR52-090210-14

105x200 cm, article no. EHR52-105200-14

105x210 cm, article no. EHR52-105210-14

120x200 cm, article no. EHR52-120200-14

Special sizes can also be produced on request.

Accessories

Wedge pillow in size 70x26x15/5 cm, article no. 7026155
Extension parts, 10 cm and 20 cm for all sizes
Transport bag

Cleaning and reconditioning

The cover should be cleaned and reconditioned to avoid any possible spread of infection. A label with washing instructions is placed on the inside of the case. The upper part of the evacuation cover is washed at a maximum of 90°C, the lower part at a maximum of 70°C. Both parts can also be tumble dried.

Use a damp cloth with a gentle detergent to clean the surface. Then wipe the damp surface with a dry cloth. If necessary, also use disinfectants such as DAX, Virkon, Perform or other commonly available disinfectants. The mattress cover can be cleaned with alcohol-based surface disinfectants containing surfactants. When cleaning with chlorine, a concentration of 1 000 ppm/0.1% can be used.

After cleaning, check the cover. If it is damaged, replace it. If the cover is damaged, also check that no liquid has penetrated the mattress core.

Checking the mattress

Check that the mattress cover is correctly positioned on the mattress. Make sure the surface of the cover is intact and inspect the seams and zipper. Adjust the bed so that your hips are level with the mattress.

Check the whole mattress, from head to foot, and make sure the mattress springs back and there is resistance. If the mattress does not spring back when the pressure is relieved, there is a risk that the pressure sharing ability has deteriorated. If you are unsure how it should feel, compare it to how a mattress feels when it is not used or by feeling at the corners of the mattress (about 20 cm from the corner) where the mattress has not been heavily loaded. It is normal for stressed areas to feel softer. The bed base should not be felt.

Destruction

Both mattress and packaging are sorted under combustible waste.

Returns and complaints

All deviations must be reported to COMCO AB. Complaints may only be made when the deviation is reported to us. Return is then done according to our instructions. If the goods have been damaged in transit, the transport company must also be notified. When the complaint is received by COMCO, the customer will be informed of how it will be dealt with.

Warranty/Life span

The Stratum Sense Evacuation Mattress has a 3-year warranty and a life expectancy of 5 years.

Other information


The mattress is stored flat, folded or rolled. If the mattress is to be stored for a longer period of time, we recommend storing it flat.

How to evacuate with Stratum Sense Evacuation mattress

- Secure the user by fastening both straps over the user.
- Inform the user what is happening.
- Lock the wheels and lower the bed to the lowest position.
- Grasp the handles at the foot of the mattress and pull the mattress down from the bed.

When evacuating via stairs

- Walk backwards until the mattress has passed the first step.
- Turn around and then pull the user by the foot end of the mattress.
- Keep a firm grip on the railing with one hand and a firm grip on the center handle with the other hand.

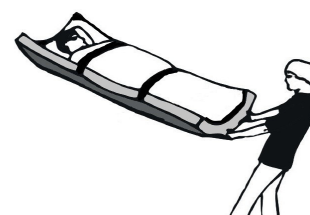

Manufacturer

Contact

Phone: +46 (0)40 45 47 87

Email: info@comco.se

Website: www.comco.se

Support

Weekdays without holidays 8.00-17.00

Postal address

COMCO AB
Brädgårdsvägen 28
236 32 Höllviken, Sweden.

Freight address

COMCO AB
Radiatorvägen, Port 5-8
283 43 Osby, Sweden.

